

PRODUCT INFORMATION SHEET

Monoclonal antibodies detecting human antigens

Anti-Hemoglobin F (HbF1)

| PURE | ASR | REF | IQP-567P | CONC | 50 µg/ml |
|--------|------------|-----|----------|------|----------|
| FITC | ASR | REF | IQP-567F | CONC | 50 µg/ml |
| R-PE | ASR ASR | REF | IQP-567R | CONC | 50 µg/ml |
| Dy-410 | ASR | REF | IQP-567D | CONC | 50 μg/ml |

Asr Analyte Specific Reagent, analytical and performance characteristics are not established.

The product comply with the ASR definitions of the U.S. Food & Drug Administration (FDA).

Description

Clone WBAC HbF1

Isotype IgG1


Specificity

The HbF1 antibody reacts with the γ -chain of the human fetal hemoglobin molecule in its intact form. Reactivity is seen with human fetal erythrocytes, adult F cells (adult erythrocytes expressing hemoglobin F), and nucleated red cell precursors containing HbF. No binding is observed with other forms of human hemoglobin, including HbA, HBA2, HBA1c, HbS, and HbE. Binding is not observed in ELISA assays for purified fetal hemoglobin alpha or gamma globulin chains alone, indicating the binding epitope is present on the intact HbF molecule only. HBF1 binding is at a different epitope on fetal hemoglobin from HBF2 clone, but has similar binding affinity as measured by flow cytometry and ELISA methods.

Application Flow cytometry

Representative Data

WBAC HbF1 was analyzed by flow cytometry on a blood sample using fetal red blood cells (RBCs) mixed with normal adult peripheral blood cells. RBCs were fixed, permeabilized and stained with anti-HbF1 FITC. The results show a good recovery of fetal blood cells in accordance with the expected value.

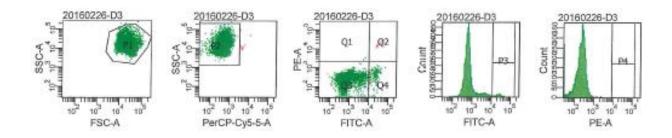

△ ♣ / * 2

Handling and Storage

Antibodies are supplied in 0.01 M sodium phosphate, 0.15 M NaCl; pH 7.3, 0.2% BSA, 0.09% sodium azide (NaN $_3$). Store the vials at 2-8 °C. Monoclonal antibodies should be protected from prolonged exposure to light. Reagents are stable for the period shown on the vial label when stored properly.

Warranty

Products sold hereunder are warranted only to conform to the quantity and contents stated on the label at the time of delivery to the customer. There are no warranties, expressed or implied, which extend beyond the description on the label of the product. IQ Products is not liable for property damage, personal injury, or economic loss caused by the product.

Characterization

To ensure consistently high-quality reagents, each batch of monoclonal antibody is tested for conformance with characteristics of a standard reagent. Representative flow cytometric data is included in this data sheet.

Warning

All products contain sodium azide. This chemical is poisonous and hazardous. Handling should be done by trained staff only.

References

Davis BH, Ben-Ezra J, Bohmer RM, Lowder JN, Mahoney WC, Sacher RA, Stuart MY, van Agthoven A: Fetal Red Cell Detection; Proposed Standard. NCCLS Document H52-A, 2000.

Explanation of used symbols

| Ti . | Consult instructions for use |
|-------------|--|
| REF | Catalogue number |
| ₹ · | Sufficient for |
| \triangle | Caution, consult accompanying document |
| * | Keep away from (sun)light |
| ፟ | Biological risks |
| * | Temperature limitation (°C) |
| ASR | Analyte Specific Reagent |
| LOT | Batch code DIIQIIL IIUOI ESCEITCE |
| | Use by yyyy-mm-dd |
| | Manufacturer |

| | Conjugates | | Ex -max (nm) | Em -max (nm) |
|-----|-------------|-------------------------------------|--------------------|--------------|
| Р | PURE | Unconjugated antibody | - | - |
| F | FITC | Fluorescein Isothiocyanate | 488 | 519 |
| R | R-PE | R-Phycoerythrin | 488, 532 | 578 |
| С | CyQ | Tandem conjugate of R-PE-and Cy5.18 | 488, 532 | 667 |
| Α | APC | Allophycocyanin | 595, 633, 635, 647 | 660 |
| D | Dy-410 | Violet Dye 410 | 405 | 460 |
| PC | PerCP | Peridinin-chlorophyll-protein | 488, 532 | 678 |
| PCC | PerCP-Cy5.5 | Tandem conjugate of PerCP-and Cy5.5 | 488, 532 | 695 |



IQ Products BV

Rozenburglaan 13a

9727 DL Groningen, The Netherlands

★ +31 (0)50 57 57 000

+31 (0)50 57 57 002

 $\begin{array}{ccc} & \text{Technical} & \underline{\text{marketing@iqproducts.nl}} \\ & \text{Orders} & \underline{\text{orders@iqproducts.nl}} \\ \end{array}$

www.iqproducts.nl